## **Bleach Home HP**

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### **Instructions for use for the patient**

Bleach Home HP is a bleaching gel for whitening discoloured teeth. It is suitable for a minimally invasive treatment using a custommade bleaching tray. The dentist regulates the treatment and determines how long it should last.

#### Areas of application

- Whitening vital, discoloured teeth
- Whitening non-vital teeth
- Whitening teeth discoloured by medication

#### Contraindications

- Known allergy to or intolerance of any of the ingredients
- Pregnancy or breast-feeding
- Heavy smoking, unless the patient abstains from smoking during treatment

#### Application

- 1 Clean the teeth thoroughly before use and clean the interdental spaces with dental floss.
- 2 Break off the cap on the tip of the syringe. Insert a small amount of gel into the hollow tooth area of the bleaching tray. One syringe is adequate to fill the upper and lower bleaching trays at least once. Partially filled syringes can be resealed with the broken off cap.
- 3 Do not fill with too much bleaching gel, as this does not improve or speed up the whitening process.
- 4 Insert the tray with bleaching gel into the mouth and place over the teeth. Do not press the bleaching tray together.
- 5 Remove excess material from the gums or palate using your finger or a toothbrush.
- 6 Wear the bleaching trays for at least 2 hours a day or overnight as instructed by your dentist.
- 7 Do not eat, drink or smoke while wearing the bleaching tray.
- 8 Remove the trays after treatment and clean them thoroughly with warm water and a toothbrush. Dry them properly and store them in the storage box. Clean your teeth and rinse thoroughly to remove any residual gel.
- 9 Continue the treatment every day as instructed by your dentist.

Avoid highly acidic food and drink (cola, citrus drinks and fruit), as these can cause tooth hypersensitivity. Strongly coloured drinks (coffee, tea, red wine) as well as smoking should be avoided during the whole treatment period, as these can retard the whitening process.

Definite whitening can normally be seen after approx. 2 weeks. In the case of stubborn staining, your dentist could extend this treatment period considerably. Depending on the tooth structure and personal habits (consuming food containing staining colouring, smoking), whitening lasts for several of years. The treatment can be repeated at a later date without any damage to the tooth structure. Consequently, the bleaching trays should be kept on completion of the treatment.

#### Information and precautions

GDF Private Label Chemistry / [www.gdfgmbh.com](http://www.gdfgmbh.com) / 23.06.2009, 0297

Tooth fillings as well as crowns, bridges and veneers made from porcelain, composite or metal are not bleached. They should be replaced if necessary and matched to the new, whiter tooth shade. There should be a period of 2 weeks between bleaching and renewing any restorations.

In certain cases there can be slight hypersensitivity of the teeth to hot and cold mucosal irritation during treatment; these symptoms disappear if the amount of bleaching gel is reduced or treatment is discontinued. Consult your dentist if the problems persist or if there are pressure points on the gums or the bleaching tray does not fit properly.

White spots may appear on the teeth during treatment. These are caused by transient demineralisation and usually disappear again.

#### Caution

Bleach Home HP contains hydrogen peroxide, glycerine, acrylic polymer, potassium nitrate, sodium fluoride and flavouring. Avoid contact with the eyes, skin and mucosal tissue. On contact, rinse with water and, on contact with the eyes, also consult a doctor.

If an excessive amount of gel is swallowed, it can cause heartburn or a feeling of nausea in sensitive patients.

#### Storage

Store the Bleach Home HP syringes in the refrigerator. Do not freeze the syringes. Protect from heat and direct sunlight. Do not use after the expiry date. Keep out of reach of children!